

CarraSmart[™] Gel

A "smart" gel that responds to wound moisture levels

Now available from Medline!







Carrington CarraSmart[™] Gel

A "smart" gel that responds to wound moisture levels

Overview

An advanced technology amorphous hydrogel formulated with glycerin and containing Acemannan™ Hydrogel for the care and pain relief of wounds. CarraSmart Gel regulates the amount of moisture present. Its intelligent response allows the gel to either donate moisture to dry wounds or absorb exudate from wet wounds. Specially formulated to adhere to wound beds and maintain optimum moisture levels for up to 72 hours.



Indications

Management of

- Pressure ulcers (stages I-IV)
- Arterial and venous stasis ulcers
- Post-surgical incisions
- Diabetic ulcers
- First and second degree burns
- Cuts and abrasions

Application

- Cleanse the wound with saline or a suitable cleanser such as Skintegrity[®] Wound Cleanser, CarraKlenz[™] or MicroKlenz[™]. Dry surrounding skin.
- 2. Apply CarraSmart Gel approximately 1/8" to 1/4" thick, covering the entire wound bed.
- Cover with an appropriate secondary dressing such as Stratasorb®.
- 4. Change as often as indicated based on the wound characteristics, usually every 1-3 days.

Removal

Carefully remove the secondary dressing and irrigate the wound as stated above. If the secondary dressing has dried to the wound edge, moisten with wound cleanser or saline until it loosens. Then remove.

Benefits

- Non-drying, allowing less frequent dressing changes.
- Smart gel responds to changing wound environment and moisture requirements (CarraSmart Gel absorbed approximately four times its weight in water while retaining a "gel-like consistency" after 48 hours).

Item Number	Description	Packaging
CRR101096	1 oz tube	24/cs
CRR101099	3 oz tube	12/cs

1-800-MEDLINE | www.medline.com

© 2001 Medline Industries, Inc. Medline is a registered trademark of Medline Industries, Inc. MKT201040/Bin119B/10M