




Operating Manual

Headrest for Conventional Wheelchairs

Assembly figure

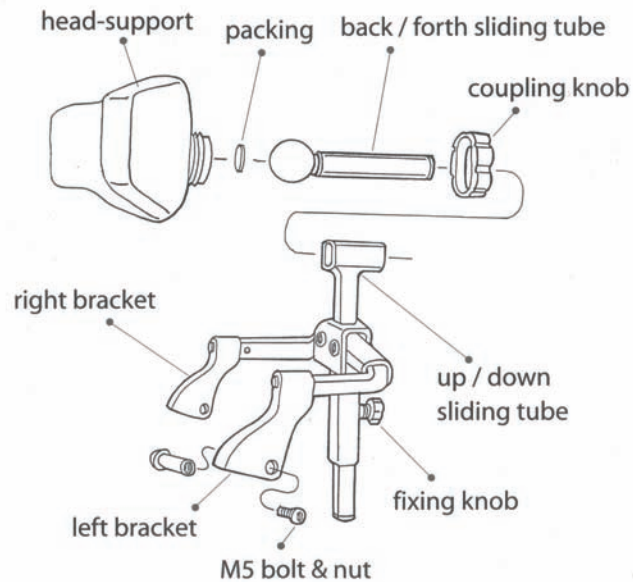

Attachments

Instructions	1 pc
M5 bolt & nut	2 sets
M5 (5m) wrench	2 pcs
M3 (3m) wrench	1 pc
Packing	1 pc





When folded

Headrest for Conventional Wheelchairs

Wheelchairs with 22 mm rear handle-tubes

Available with:



Large: for wheelchair seat width between 42 cm-50 cm

Medium: for wheelchair seat width between 36 cm-44 cm

INSTRUCTIONS

This headrest is designed to be used only as indicated in the pamphlet. The manufacturer can not be held liable for any injuries sustained or damage to the wheelchair due to modification made to this device.

To avoid injuries or possible damage to the wheelchair, please read the instructions carefully before installing this device.

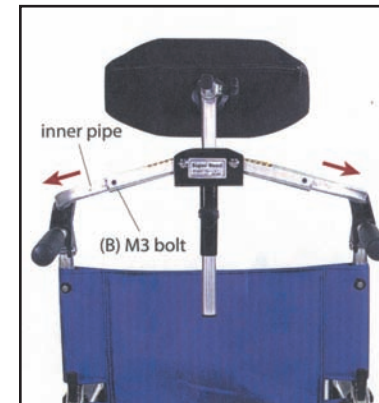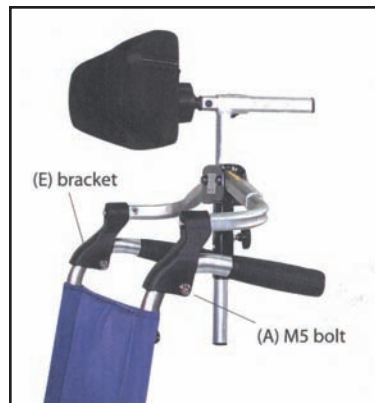

How to install:

Unfold wheelchair first, loosen bolts (B) to allow inner tubes to slide freely. Fit brackets (E) onto the curved part of the push-handles, then tighten bolts (A)

Have a person help by slightly pushing rear handles of wheelchair apart, simultaneously adjust center joints as high as possible (in order to obtain adequate support tension for headrest), then tighten bolts (B).

Make certain all the fittings on the headrest are tight and secure before using.



How to use:

When using wheelchair, unfold wheelchair first, then prop up the headrest.

Conversely when folding wheelchair, first pull down the headrest to release the support tension, then fold wheelchair.

WARNING: DO NOT put fingers into the joints of the headrest or injury could result. Do not use headrest as a restraint. Do not use in a moving vehicle.