



## Medline Non-Adherent Impregnated Gauze Dressings

<u>Type of Dressing</u>	<u>Indications</u>	<u>Contraindications</u>	<u>Advantages</u>	<u>Disadvantages</u>	<u>Frequency of Dressing Change</u>	<u>Preferred Outcomes</u>	<u>Name of Dressing</u>	<u>Sizes</u>	<u>Item Numbers</u>
<b>Impregnated Non-Adherent -- Petrolatum Gauze</b>	The petrolatum gauze dressing is intended for use as a primary contact layer in dressing wounds such as minor burns, skin grafts, skin donor sites, as an initial layer in surgical dressings, as a protective layer for injured body sites and a seal for tubes and drains leading from the body.	Third degree burns, long term, permanent or no change dressing use. Heavily exudating wounds.	soothing, protects wounds/skin from trauma of secondary/cover dressings	none	As required by the users' normal protocol.	maintains moisture, doesn't allow secondary dressings to "stick" to wound/periwound skin to prevent trauma with dressing changes, permeable to allow minimal drainage to "pass through" to secondary dressing	Petrolatum Gauze	1"x36" 1"x8" 1/2"x72" 3"x18" 3"x36" 3"x9" 6"x36" Overwraps available	NON251136 NON251180 NON251272 NON252272 NON251318 NON252318 NON251336 NON251390 NON251636
<b>Impregnated Non-Adherent - Xeroform Petrolatum Gauze Dressing</b>	The Xeroform Petrolatum Gauze dressing is intended for use as a primary contact layer in dressing wounds such as lacerations, skin graft recipient sites, newly sutured wounds, abrasions and minor or partial thickness burns. It may also be used as an initial layer in dressing surgical wounds with light exudate where mild medication and deodorization are desired.	Third degree burns, as a long term, permanent or no change dressing. Heavily exudating wounds.	soothing, protects wounds/skin from trauma of secondary/cover dressings, bacteriostatic	none	As required by the users' normal protocol.	maintains moisture, doesn't allow secondary dressings to "stick" to wound/periwound skin to prevent trauma with dressing changes, permeable to allow minimal drainage to "pass through" to secondary dressing	Xeroform Gauze	1" x 8" 2" x 2" 4" x 4" 5" x 9" Overwraps available	NON253180 NON255180 NON253220 NON253440 NON253590
<b>Impregnated Non-Adherent - Oil Emulsion Dressing</b>	The Oil Emulsion dressing is intended for use as a primary contact layer for exudating wounds beneath an absorbent secondary dressing thus minimizing maceration. The dressing should be used on any draining wound including skin ulcers, burns, infected areas, suture lines, lacerations, colostomies and episiotomy incisions.	Third degree burns or as a long term, permanent or no change dressing.	soothing, protects wounds/skin from trauma of secondary/cover dressings, maintains moist wound bed for autolytic healing	none	As required by the users' normal protocol.	maintains moisture, doesn't allow secondary dressings to "stick" to wound/periwound skin to prevent trauma with dressing changes, permeable to allow drainage to "pass through" to secondary dressing	Oil Emulsion	3" x 3" 3" x 8" 3" x 16" 3"x8" 3 dressing/pkg	NON250330 NON250381 NON250317 NON250383