



Reorder No.: MDS89745R

BATH BENCH WITH BACK

Assembly

1. Orient legs as shown in figure 1.
Insert screw and lock-washer through legs and fasten with nut. **Do NOT** tighten them fully.
2. Lay bench seat upside down and align leg assembly from Figure 1 with the leg tube channels on the bottom of the seat as shown in Figure 2.


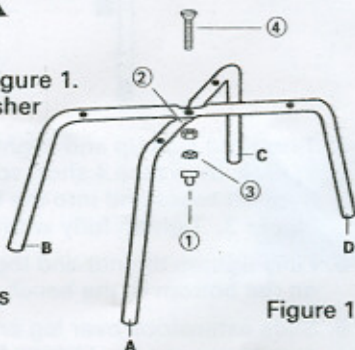


Figure 1


Make sure the letters ABCD are aligned and match the letters on the bottom of the plastic seat before fixing them.

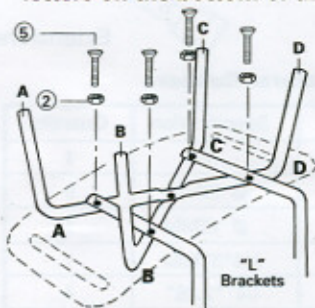

3. Insert the 4 long screws provided through the shorter side of the "L" bracket and through the legs and into the bench seat (Figure 2)
Only hand tighten screws at this point.


Figure 2

Figure 3

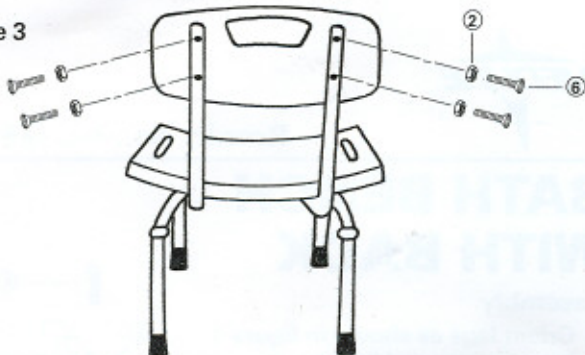

4. Turn right side up and align "L" bracket with channels in back. Insert the 4 short screws through the back. Support tubes and into the backrest as shown in Figure 3. Tighten fully with screwdriver.
5. Fully tighten the nut and the long screws located on the bottom of the bench.
6. Slide extensions over leg ends making sure to engage the locking button.
7. Make sure the bench is securely assembled prior to using.

Figure 4

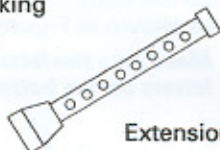

Extensions

This Product Contains Dry Natural Rubber.

No.	Description	Specification	Quantity
①	M6 NUT WITH CAP	M6	1
②	STAINLESS STEEL WASHER	Ø 13MM	9
③	GEAR-SHAPE WASHER	Ø 12MM	1
④	37 CROSS SCREW	M6*37MM	1
⑤	CROSS SCREW	1/4" * 2.25"	4
⑥	CROSS SCREW	1/4" * 1.5"	4